


Mini-Porter-Cyclone MP-19

Instructions


① Please make sure the following parts are contained inside the package:

Mini-Porter-Cyclone.....	1
3L Transparent Tank	1
Suction Pipe.....	1
Antistatic Hose.....	1
Hose Clamp	2

② Please check whether the packing (Black) is attached well with the 3L transparent tank. Put the Cyclone on the top of the transparent tank and fasten with the draw latches.

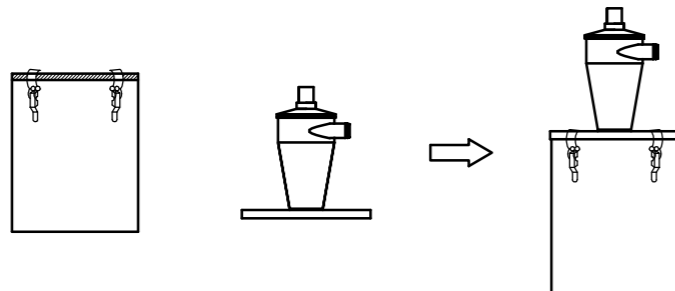

③ Attach the antistatic hose with the suction mouth on the side of the Mini-Porter-Cyclone and fasten well with the hose clamp. (It can be made easier by turning the hose clockwise while attaching.)

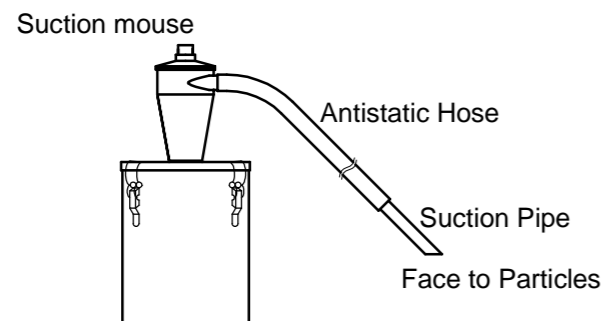

④ Connect the suction pipe with the antistatic hose and fasten well with the hose clamp. (It can be made easier by turning the hose clockwise while attaching.)

⑤ Connect the antistatic hose of the Silent-Cleaner (our products) with the suction mouth on top of the Mini-Porter-Cyclone.

⑥ Suction will start as soon as the vacuum cleaner is switched on. Then start the suction operation after waiting for 2 or 3 seconds till the airflow stabilizes. Vacuum the particles with the air when operating.

★ Caution

- Please inspect and clean the transparent tank occasionally. If the transparent tank fills up, the vacuumed dust may enter into the cleaner.
- Please ground/ earth the device to control static electricity.




TEL: +81-6-6572-1207

FAX: +81-6-6572-1256

E-mail: osawa@osawa-company.co.jp


In case you have any inquiry, please contact the dealer or the manufacturer.

Cleaning Maintenance

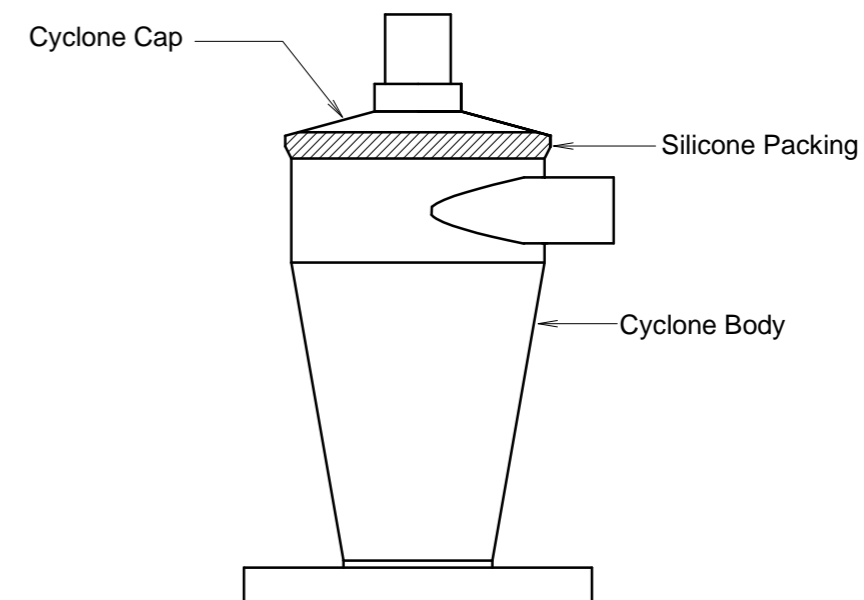

※ Frequent maintenance is not needed for Mini-Porter-Cyclone. However, occasional cleaning is required inside of the body to maintain optimal performance. To clean, brush the inside occasionally. Especially after vacuuming the moist dust, brush well or wipe well with the dry cloth since the moist dust stays inside.

How to take off the Cyclone Cap

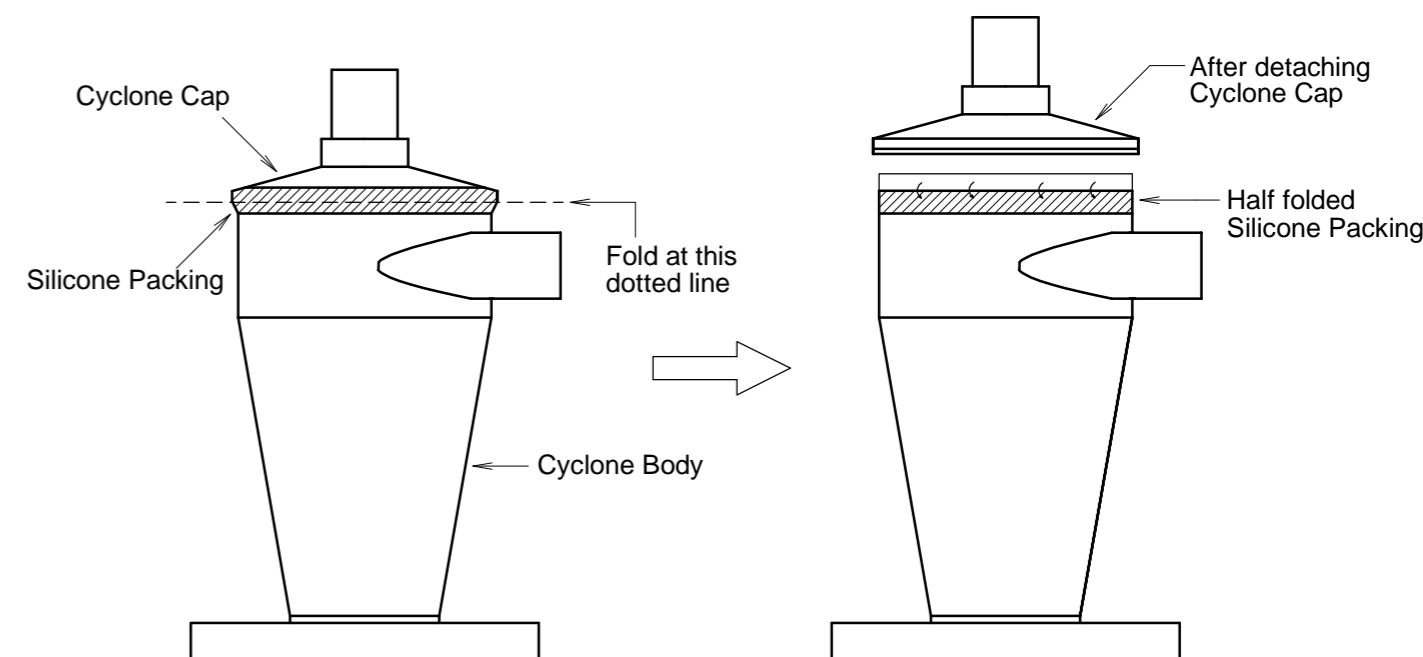

- ① Take off the antistatic hose.
 - ② Fold the silicone packing as shown in the drawing and take off the Cyclone Cap. Silicone packing should be kept half folded on the Cyclone Body.
- ※ To attach the Cyclone Cap, follow the reverse order of taking off. When attaching, please make sure that the body and Cyclone Cap are not covered with any dust.

How to dispose the dust

- ① Switch off the vacuum cleaner.
- ② Disconnect the hose from Mini-Porter-Cyclone.
- ③ Undo the 3 draw latches slowly. Please be careful when detaching the Mini-Porter-Cyclone and the tank since the dust inside the tank may get scattered.